

Fitted Surgical Mask out of N99


Do not use pins anywhere on this fabric except within seam allowances.

The fabric is two layers, one white side and one blue side.

Take pattern and cut one and then flip pattern and cut another one. This will give you 2 white pieces for the inside of the mask and 2 blues for the outside of the mask.

Take the two blue and put right sides together and do same for white.

Stitch the center seam with $\frac{3}{8}$ " seam allowance and then clip to allow ease.

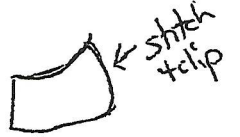

Cut 2 6" long pieces of $\frac{1}{8}$ " or $\frac{1}{4}$ " elastic.

Come in about $\frac{1}{2}$ " and sew to right side of the blue piece. Needs to be a little less than $\frac{3}{8}$ " seam so that your stitching doesn't show when we turn it.



Pin the right sides of the mask together. Only use a few pins and make sure within the seam allowances.

Start stitching at lower part of mask and go around entire mask but leave a fairly large opening at bottom to allow turning later.



Cut 8" piece of floral wire or something comparable.

Use zig zag stitch to attach it within the seam allowance of the nose side.



Turn mask so right sides out and finger press. DO NOT USE AN IRON Probably will need a dull point to poke into the corners.

Topstitch the lower edge in order to close the opening.

Congrats, you have just finished your mask.

